UPUTSTVO ZA UPOTREBU (SRB)

Sabouraud Chloramphenicol Agar Plate

Podloga preporučena za selektivnu kultivaciju kvasaca i plesni.

Sadržaj pakovanja:

Šifra artikla (pakovanja) REF	Opis	Šifra primarnog pakovanja:	Broj podloga
PRM1067V20	Podloga izlivena u petri posudama od ø90	PRM1067	20
PRM1067V60			60
PRM1067V240			240
PRM1067M40	Podloga izlivena u petri posudama od ø50		40

Uputstva

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INSTRUCTION FOR USE (EN)

Sabouraud Chloramphenicol Agar Plate

Medium is recommended for selective cultivation of yeasts and moulds.

Package contents:

Item code (packaging) REF	Description	Primary packaging code:	Number of products
PRM1067V20	Substrate poured into petri dishes of ø90	PRM1067	20
PRM1067V60			60
PRM1067V240			240
PRM1067M40	Substrate poured into petri dishes of ø50		40

Directions

Surface spread the test inoculum aseptically on the plate.

Principle and interpretation

Sabouraud Chloramphenicol Agar is cited as Medium C and recommended for cultivation of yeasts and moulds. This medium was described originally by Sabouraud (1) for the cultivation of fungi, particularly useful for the fungi associated with skin infections. The medium is often used with antibiotics such as Chloramphenicol (2) for the isolation of pathogenic fungi from materials containing large numbers of fungi or bacteria. Casein enzymic hydrolysate and peptic digest of animal tissue provide nitrogenous compounds. Dextrose provides an energy source. Chloramphenicol inhibits a wide range of Gram-positive and Gram-negative bacteria which makes the medium selective for fungi (3). The low pH favours fungal growth and inhibits contaminating bacteria from clinical specimens (4).

Quality control

The data and results of quality control are given in the certificate of analysis for each lot.

Storage and shelf life

Storage between 15-25°C. After opening storage between 2-8°C. Use before expiry date on the label.

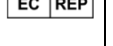
Warning and precautions

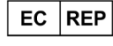
In vitro diagnostic use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques.

Symbols used on labels

	European Conformity mark		This side up
	is an in vitro diagnostic medical device (IVD)		Catalogue number
	Do not expose directly to sunlight		Batch code
	Consult instructions for use		Use-by date
	Do not re-use		Temperature limit
	Pack size		Manufacturer
	European Authorized Representative (Authorised Representative)		

	Salus Cons kft. 6722 Szeged, Bécsi krt 23, HUNGARY e-mail: office@saluscons.com
---	--

Reference

1. Sabouraud K., 1892, Ann. Dermatol. Syphilol, 3:1061.
2. Ajello L., 1957, J. Chron. Dis., 5:545.
3. Lorian (Ed.), 1980, Antibiotics In Laboratory Medicine, Williams and Wilkins, Baltimore.
4. Murray, P. R 2005, In Manual of Clinical Microbiology